

# Footplate Riser, 2"-9" Lower Extension, Adj. Center Mount / Sing. Act. AFP

**Please note:**

1. The complete assembly does not contain the calf panel or footplates.
2. The footplate riser can only be used with the adjustable center mount foot platform or the single actuator AFP. It cannot be used with the basic center mount foot platform or HD / dual actuator AFP.
3. The achievable extension length varies between the center mounts and AFP. See the mounting hole matrix below.
4. The standard calf pads and brackets must be removed in order to mount the footplate riser (not shown).
5. The footplates will need to be removed from the tube style lower extensions in order to mount them onto the footplate riser. See the tube style lower extension & footplate page for selection.
6. The calf panel can only be used if the length is set from 6"-9". If the length is set from 2"-5" the calf panel cannot be used.

