



# CP COMFORT PLUS SERIES



# HEAD SOLUTIONS

# CP SERIES OVERVIEW

The Comfort Plus original is our most popular and affordable contoured head support. The CP Series includes the **Original**, the **Adjustable**, and the **Tri-Comfort Plus**. The generic contour of the **Original CP** encourages a midline head position. The **Fixed** or **Adjustable** angled pads provide more aggressive lateral support, as needed.

The dual layered foam structure reduces pressure when the posterior head is bearing weight, such as during tilt or recline. Wider options can limit **hooking** under the pad.

## COMFORT PLUS CONFIGURATIONS

**CP**


ORIGINAL CONTOURED



**Comfort Plus**  
**Original** Contour

**ACP**

ADJUSTABLE COMFORT PLUS

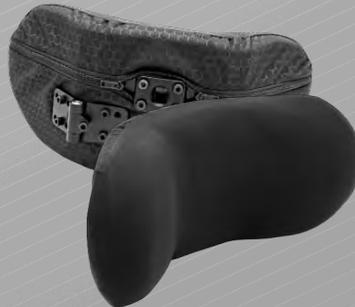

**Comfort Plus** with  
**Adjustable** Laterals

**TCP**

TRI-COMFORT PLUS



**Comfort Plus** with  
**Fixed** Laterals



# CP SERIES DETAILS SERIES FEATURES

## GENERAL FEATURES THROUGHOUT THE SERIES

### DUAL LAYERED FOAM RIGID METAL FRAME



Contact layer is a 1½ in (3.81cm) open cell contact foam

Base layer is a 5/8 in (15.88mm) latex free closed cell foam base

Foam is applied to the base plate with a hook & loop material for easy removal. Making it easy to replace or wash.



### COOLCORE® COVER

The CP Series design has been refreshed with a 17mm ball configuration and now comes standard with Coolcore covers.

Coolcore is a research proven performance fabric technology that adapts to its environment and regulates temperature by managing heat and moisture.

The CP Series' covers can be replaced with a wide variety of cover materials to choose from, like our traditional Neoprene cover, Glide Skin and more. Scan OTHER COVER OPTIONS code below to see all available.

Got questions? 1 (800) 965-9229 • 1 (512) 715-9995


A complete range of foams, gels, and memory foams materials are available to add an extra level of personality and comfort. Here are some examples.

#### Flat Gel


Gel inserts can be added on the inner surfaces for clients who exert strong forces.




#### SunMate® FRG Foams

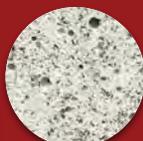


Medium Soft

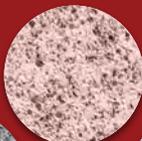


Soft

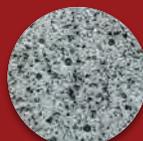

Extra Extra Soft

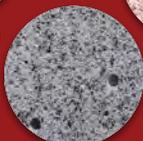

Medium

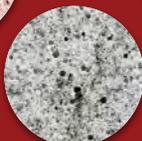

Extra Soft



MORE ABOUT  
Coolcore




OTHER COVER  
OPTIONS

# COMFORT PLUS SERIES FIXED LATERALS

## TRI-COMFORT PLUS 45° FIXED LATERALS

The **Tri-Comfort Plus (TCP)** expands the features of the Comfort Plus with the addition of angled lateral wings and a clinically based choice of length and heights. Increase lateral control with side wings that offer help in keeping the head on the center posterior support.

### TRI-COMFORT PLUS SIZE OPTIONS

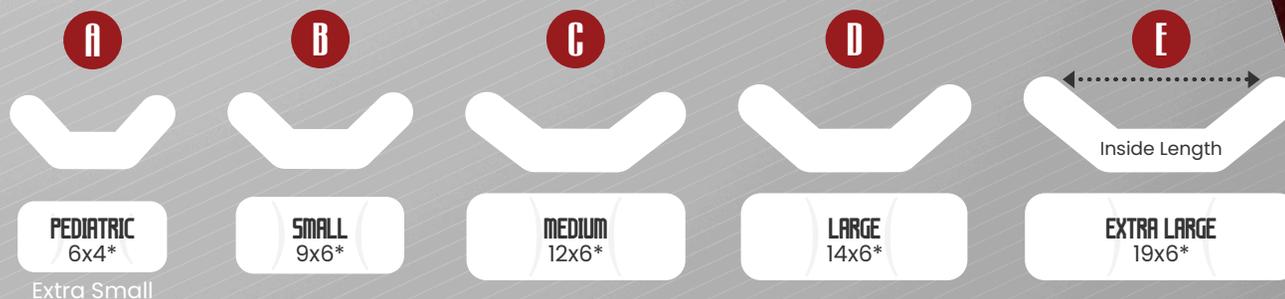

### Customizations

- ✓ Foam Material
- ✓ Cover Material
- ✓ Pad Dimensions

	I.N.	Description	Final Width	Final Height	Inside Length	Wing's Length
<b>A</b>	TCP505M	<b>Pediatric</b> TCP	7¼in (19.69cm)	4½in (11.43cm)	4¾in (12.07cm)	3in (7.62cm)
<b>B</b>	TCP405M	<b>Small</b> TCP	8¼in (22.23cm)	5½in (13.97cm)	6¼in (16.51cm)	3½in (8.89cm)
<b>C</b>	TCP305M	<b>Medium</b> TCP	11½in (29.21cm)	6in (15.24cm)	8½in (21.59cm)	3¾in (9.53cm)
<b>D</b>	TCP205M	<b>Large</b> TCP	14in (35.56cm)	6in (15.24cm)	11in (27.94cm)	5¼in (13.34cm)
<b>E</b>	TCPI05M	<b>Extra Large</b> TCP	18½in (46.99cm)	6in (15.24cm)	15¼in (38.74cm)	7¼in (18.42cm)

\*Dimensions shown with pad icons are rounded final dimensions. This table displays the **true** final dimensions which include base, pad and cover.

# COMFORT PLUS SERIES MODIFICATIONS

**CUSTOMIZATIONS  
AVAILABLE FOR CPS**

## EMBEDDED SWITCHES

- ✓ CP
- ✓ ACP
- ✓ TCP

Switches can be embedded into the pad for a clean and discreet look. Either mechanical or proximity switches work with any after market alternative control system or our **Award Wining i-Drive**.

Switches can be used to drive, control tilt/recline or to operate devices like phones and desktops. Switches comes in many formats and shapes like **mechanical** switches and **proximity sensor** switches, for example **Capacitive** or **Fiber Optics**. Some switches cannot be embedded. Contact a representative for more information.

Sip & Puff mechanical switches can be integrated as well with our Swing Away Receiver, Standard or 4 point.




## FULL PAD CUSTOMIZATION

- ✓ CP
- ✓ ACP
- ✓ TCP

The **Original (CP)** and the **Fixed Angles (TCP)** can be customized to fit the customer. From cover, dimensions, and foam to gel additions.

Dimensions provided will be applied to the contour metal frame which will be padded with an approximated **1/4in (6.35mm)** offset around the frame to avoid contact with the cover or client. Contact a Stealth representative or one of our customer service representative members for more information.

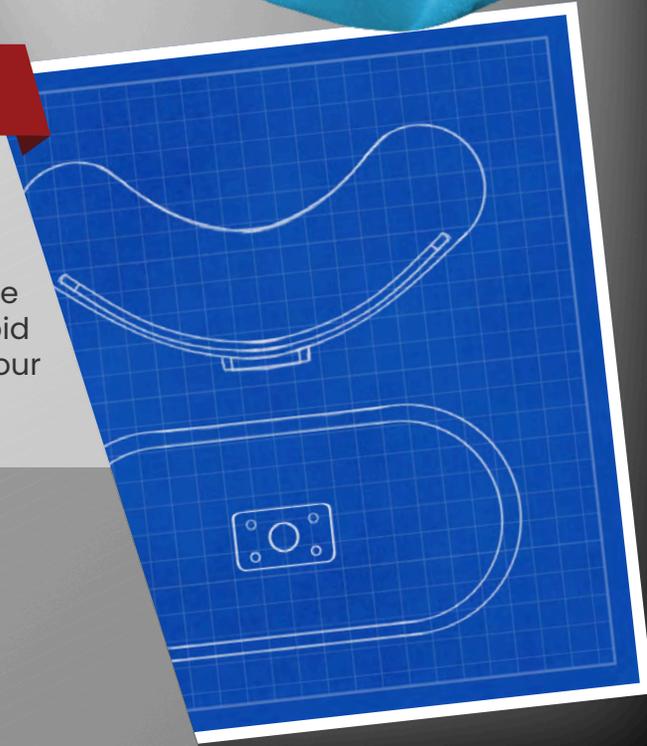

Got questions? Call us at one of the phone numbers below

**1 (800) 965-9229 or 1 (512) 715-9995**

or contact us at [information@sw.stealthproducts.com](mailto:information@sw.stealthproducts.com)

# COMFORT PLUS SERIES CPS ADD-ONS

**CUSTOMIZATIONS  
AVAILABLE FOR CPS**

## SWING AWAY RECEIVERS



**SAR4A-DB**




**SAR2A-B**

The **Comfort Plus Series (CPS)** is compatible with our Swing Away System **Standard (SAR2A)** and **4 point (SAR4A)**. The swing away system helps you place pads, switches, sensor switches, sip & puff, alternative driving methods, or lateral support where it is needed most.

Either to enhance positioning or to complement alternative control options, the Swing Away will let a caretaker or technician configure the Combo to offer solutions to any challenge.

## ALTERNATIVE CONTROL TECHNOLOGIES

The **Comfort Plus Series** can be used as a support for tilt or recline. It can supplement with embedded switches on a tray driving control configuration, or any other configuration. It is compatible with our **i-Drive®** Advanced Alternative Control System and **i-Connect™** sensor switches power router.

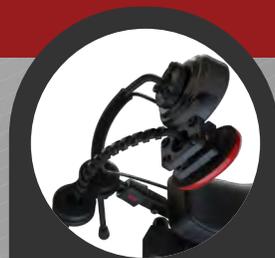

# COMFORT PLUS SERIES COMPARISON CHART



Feature	Comfort Plus	Tri-Comfort Plus	Adjustable Comfort Plus
Format	Curved Frame	Angled Frame (120°)	Tree Part Frame. Center Pad with Adjustable Wings
Positioning Features	Support Only	Aggressive Lateral Support	Custom Lateral Support
Sizes	P S M L XL	P S M L XL	14in (35.56cm) or 19in (48.26cm) Flat
Customizable	Yes	Yes	Yes
Lateral Support	With Swing Away Add-on	Yes	Yes
Swing Away Compatible		Standard (High Profile)	
Foam Structure		Dual Layered	
Tilt / Recline Support		Yes	
Standard Cover		Coolcore	
Cover Options		All Available	
Washable Cover		Yes	
Removable Foam		Yes	

\* if further customization needed, contact your Stealth Products' international representatives or our Customer Service Representative.