

- Please Note:**
1. Due to a product enhancement, as of Serial Number **JF301123108020**, the iLevel® base components (A1 - B3) and cable block (WL1) have been reconfigured to use the spring and retainers (A8 - A10).
    - A. If replacing the iLevel® base components (A1 - B3) on a unit manufactured prior to **JF301123108020**, the spring and retainers (A8 - A10) are not utilized and the cable connects to the suspension limiter using the clevis pin.
    - B. If the cable block (WL1) is replaced with the new version on a unit manufactured prior **JF301123108020**, the spring and retainers (A8 - A10) must be utilized. The new cable cannot mount without them.
  2. As of Serial Number **JF309425032020**, the retainers (A8) and (A10) have been reconfigured to a new version.
    - A. If replacing (A8) or (A10) a unit manufactured between **JF301123108020** to **JF309425032020**, (A8) and (A10) must be replaced together **on both sides**.

