

- Please Note:**
1. Due to a product enhancement, as of Serial Number **JF301123108020** the iLevel® Cable and Cable Block assembly (A1) has been replaced with a new version.
    - A. If replacing the iLevel® Cable Block assembly on a unit manufactured prior to **JF301123108020**, the spring and spring retainers must be added to both side (**C1 x 2, D1 x 2, and E1 x 2**). The new cable block and cable assembly will not mount to the suspension limiter without the spring and retainers.
    - B. The new cable no longer utilizes steel tubing, rather, the cable is flexible and can be curved and routed to shape.
    - C. When installing the new cables, the cables should cross in the front as shown.
  2. As of Serial Number **JF309425032020**, the retainers (E1) and (C1) have been reconfigured to a new version.
    - A. If replacing (E1) or (C1) on a unit manufactured between **JF301123108020** and **JF309425032020**, (E1) and (C1) must be replaced together **on both sides**.

